



KN95CONEV

Cone-Shaped KN95 Respirator with Filter Valve

Features

- CE, FDA, KN95 certified, with 95% filtration against bacteria and particulates
- The cone-shaped KN95 with filter valves let hot air out more easily, which makes breathing more comfortable and reduces fogging of glasses/goggles.
- Cone shape can contour to the face for added comfort
- Economical, light, environmentally-friendly
- Adjustable nose wire for extra comfort
- Soft foam nose cushion for an easy and comfortable fit

Applications

- Personal protection
- Food processing
- Construction

Technical Specifications

Material: Non woven fabric, Cotton, Meltblown

Color: White mask & valve, red headbands

Filter Performance: > 95%

Test Agent: NaCl

Flow Rate: 85 L/min

Total Inward Leakage: < 8%

Inhalation Resistance: < 30 Pa

Exhalation Resistance: < 250 Pa

Exhalation Valve Leakage Requirement: Depressurization to 0 Pa > 20 sec

Force Applied: - 1180 Pa

CO2 Clearance Requirement: < 1%

Sizes: One Size Fits Most

Packaging: 10 masks per box; 20 boxes per case

Country of Origin: China

Donning Instructions

Step 1: Hold respirator in your hand, with the nosepiece at your fingertips and the headbands hanging freely below your hand.

Step 2: Place respirator under your chin with the nosepiece up. Pull the bottom headband overhead and place around neck below ears.

Step 3: Pull the top headband to top back of head.

Step 4: Adjust headbands for comfortable fit. Push the nosepiece inward with your fingertips to conform to the shape of your nose.

Step 5: Cup both hands over the mask, checking for tightness, inhale sharply, negative pressure should be felt in the mask, re-adjust as necessary.