



## ORAL THERMOMETER

Digital Oral Thermometer For Adults & Children

### Features

- Easy to read digital LCD display.
- Ideal for family usage.
- Accurate temperature measurement with beep prompt
- Can be used orally, rectally and under the armpit.
- Rigid steel tip
- Easily switch between Celsius (°C) or Fahrenheit (°F)
- Transparent protective storage case
- Automatically power off within 10 minutes after non-use
- Fever alarm for temperatures higher than 100.04 °F (37.8 °C)

### Applications

- Family/Home
- Schools
- Hospitals
- Clinics
- Workplaces

### Technical Specifications

**Measuring range:** (89.6 °F-109.4 °F)/(32 °C-43 °C)

**Accuracy:** ± 0.1 °C (between 35.0 °C and 39.0 °C);  
± 0.2 °C (lower than 35.0 °C and higher than 39.0 °C)

**Response:** 60 seconds

**Memory:** Last measuring result

**Battery:** One 1.5V(LR41) button battery

**Dimension:** 5 x 1 x 0.4 in (LxWxH)

**Weight:** 0.5oz (15g)

### Usage Instructions

*A digital thermometer can be used three different ways. These include:*

- **Oral:** For this method, the thermometer is placed under the tongue. This method is used for adults and children 4 and over who are able to hold thermometer in mouth.
- **Rectal:** For this method, the thermometer is inserted gently into the rectum. This is mostly done in babies but can be used in children up to 3 years of age. You can take rectal temperatures in children older than 3 years, but it might be difficult to keep them as still as they need to stay.
- **Axillary:** For this method, the thermometer is placed in the armpit for young children or adults whose temperature can't safely be done orally. This method is not as accurate as oral or rectal but can be used as a quick first check. You can follow this with an oral or rectal reading.